GR EN

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INTENDED USE:

This product is intended to be used as a waterproof plaster or bandage protector in the shower or bath.

IMPORTANT:

- Before use, lightly file any sharp edges of the plaster that may pierce the protector.
- Hold the plastic ring firmly and then place the hand or foot that have the plaster or bandage inside.
- Make sure the protector adequately covers the plaster or bandage.
- Dry the plastic ring before removing the protector.
- When removing the protector, hold it by the plastic ring and not by the PVC bag as product failure may be caused.
- The product is intended for single patient use.

CAUTION (!):

- Do not use sharp objects in the product.
- Protect the product from abrasions, cuts and holes.
- Plastic ring is not designed to have heavy weight applied to it.
- .The protector can become slippery while wet, as the surroundings in the shower area. Use extreme caution when bathing or showering in these conditions.
- Make sure the ring is not too tight in the arm or leg area. Consult your doctor if the ring causes swelling or discomfort.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

ENVIRONMENTAL PROTECTION:

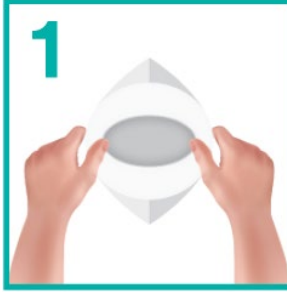
If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

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Stretch out the silicone hole for limb entry.

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Carefully insert the cast or bandaged part of the limb.

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Completely cover the cast or bandage. Ensure the silicone hole is properly sealed to the skin and straighten out the silicone film.

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WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by the dealer.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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