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Guidance and declaration of manufacturer -Electromagnetic emission	
Immunity test	Compliance
Electrostatic discharge (ESD) IEC 61000-4-2 ISO7176-21	±8 kV contact ±15 kV Air
Electrical fast transient/bursts IEC 61000-4-4 ISO7176-21	±2 kV for power supply lines
Surge IEC 61000-4-5 ISO7176-21	±1 kV line to line
Voltage dips, short interruptions and Voltage variations on power supply input lines IEC 61000-4-11 ISO7176-21	0% U <sub>i</sub> 0.5 cycle at 0°, 45°, 90°,135°, 180°, 225°, 270° and 315°
	0% U <sub>i</sub> 1 cycle 70% U <sub>i</sub> 25/30 cycles at 0°  Voltage short interruptions : 0% UT 250/300 cycles at 0°
Power frequency (50 Hz) magnetic IEC 61000-4-8 ISO7176-21	30 A/m
Radiated RF EM fields IEC 61000-4-3 ISO7176-21	20V/m 80MHz - 2.7GHz 80% AM at 1kHz
Conducted disturbances induced by RF fields IEC 61000-4-6 ISO7176-21	3V/m 0.15MHz - 80MHz 6V in ISM and amateur radio bands between 0.15MHz and 80MHz 80% AM at 1kHz
Note: U <sub>i</sub> is the a.c. mains voltage prior to application of the test level.	

Electric and magnetic environment guidance in use	
Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Emission of harmonics IEC 61000-3-2	NA
Voltage fluctuations / flicker emissions IEC 61000-3-3	NA

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment						
Test frequency (MHz)	Band (MHz)	Service	Service	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 - 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27
450	430 - 470	GMRS 460, FRS 460	FM $\pm 5$ kHz deviation 1 kHz sine	2	0,3	28
710	704 - 787	LTE Band 13,17	Pulse modulation 217 Hz	0,2	0,3	9
745						
780						
810	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0,3	28
870						
930						
1720	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1,3,4,25; UMTS	Pulse modulation 217 Hz	2	0,3	28
1845						
1970						
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28

5240	5100 - 5800	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28
5500						
5785						
NOTE If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.						

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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## Preface

Please read the user manual carefully before using the product.

- ✧ This manual contains functions, assembly methods and simple troubleshooting.
- ✧ This manual is valid for **0811579**
- ✧ This manual contains wheelchair maintenance and inspection methods. Please place it in a conspicuous place so that you can easily refer to it when you need it.
- ✧ Give this manual for reference when other people are going to use this wheelchair.
- ✧ The annotations and images in this manual may differ slightly from the actual parts due to quality improvement or design change. Please check it carefully.
- ✧ The wheelchair you have in your hands may differ from the one in the following photos. The assembly process is similar.
- ✧ Contact your dealer if there is any doubt or question.
- ✧ Improper use of any vehicle may result in injury. Unsafe driving could harm yourself and others.
- ✧ The power wheelchair is intended to comfortably transport people with walking difficulties or no walking ability.
- ✧ The power wheelchair is designed to transport only 1 person.

Symbols used in this manual

Warning symbol

Follow the instructions next to this symbol carefully.

Failure to pay attention and follow the instructions may result in personal injury or damage to the wheelchair or the environment.

# 1.Product description

## Product display

## Product specification

Frame	Aluminum- Foldable
Controller	Right or Left Adjustable Controller
Motor	130 W x 24 V x 2
Battery	2x24V-5.2 Ah, lithium
Charger	160V~240V , 50/60Hz, 29.6V, 3A
Max Speed	6 km/h
Autonomy*	≥20km
Gradeability	6°
Front wheel	7", PU solid
Rear wheel	12", pneumatic
Armrest	Flip back, with PU cushion
Footrest	Reversible
Seat & Backrest	Oxford fabric with linen cover, with flame retardant properties
Brake	Electromagnetic brake
Protection class	IPX4
Brake performance in horizontal plane	≤1.5m
Obstacle clearance height	≥40mm
Obstacle width	100mm
Minimum turning radius	1.2m
Other features	Safety belt, Folding Backrest, anti-tilt wheels, audible warning-alarm when reversing, Storage compartment on the back, horn

**\*Approximative autonomy as also determined by the weight of occupant, the battery condition, the ambient temperature, the way of driving, the terrain and the wind speed.**

											<b>GROSS WEIGHT</b>
43 cm	43 cm	39 cm	40 cm	65 cm	91 cm	89 cm	47 cm	12" 7"	120 Kg	15* Kg	22 Kg

**\* Net weight without batteries. The weight including batteries is 17 Kg.**

### **Product Performance**

This series of motorized electric wheelchairs is powered by a battery. Users control the direction and adjust the speed with the joystick. It is suitable for low speeds with smooth road surface and gentle slope.

### **Main Structure**

This series of electric wheelchairs is mainly composed of two parts:

- 1) Frame part: frame, wheel, footrest, armrest, backrest, etc.
- 2) Electrical part: controller, motor, battery.

### **Scope of Application & Intended Use**

The electric wheelchair is suitable for people with disabilities and the elderly who use it as a vehicle for transportation. It is intended for indoor and outdoor use. For outdoor use, it is suitable for medium distances. It is not suitable for use on highways.

## **2. Safety instructions**

### **Driving warnings**

#### **General driving warnings**

- ♦ Keep your hands on the armrests to control the wheelchair.
- ♦ Practice in parks or other safe open spaces until you can use the wheelchair skillfully.
- ♦ Practice driving in a safe place so that you can move comfortably in all directions.
- ♦ Strictly follow traffic rules for pedestrians.
- ♦ Maintain a steady straight course when driving and avoid sharp turns.
- ♦ Maintain the correct tire pressure. Incorrect tire pressure can cause unstable driving or increased battery power loss.

#### **Accompanied by caregivers or avoiding driving under these conditions**

- ♦ Driving in bad weather conditions, such as rain, heavy fog, strong wind, snow, etc.
- ♦ In case the wheelchair gets wet, wipe off the water immediately.
- ♦ Driving on an uneven road with mud, sand, gravel, etc.
- ♦ Driving on a road with traffic.
- ♦ Driving on a road without roadside guards.
- ♦ Driving near railway tracks.
- ♦ When you have to cross a railway, make sure it is safe and that the tires will not stick to the rails.
- ♦ The electric wheelchair is for personal use only, it does not transport people or goods and is not intended for towing.

#### **Precautions for uphill and downhill driving**

- ♦ Avoid driving on steep slopes. The slope range should be up to 9 degrees.
- ♦ Keep moving forward when going uphill or downhill
- ♦ Keep your speed low when going downhill
- ♦ Do not drive on steps
- ♦ Avoid potholes

#### **Precautions for caregivers**

- ♦ Caregivers should confirm that the users' feet are in the correct position on the footrests.
- ♦ For the user's safety, caregivers should operate the wheelchair in manual mode in cases of steep roads.

## Other information

### Repair and modification

If repair and modification is required, please contact your dealer. Do not make modifications yourself, it may lead to accident or malfunction.

### Protect the environment

To protect the environment, do not dispose of the wheelchair and battery. Contact your dealer for a new battery replacement.

### Keep the storage environment dry

Do not store the wheelchair in a humid environment such as a bathroom. If the wheelchair gets wet, wipe off the water immediately.

*Do not replace parts or materials at random*

*Do not add weight at random, so as not to cause imbalance*

*Do not use another vehicle to pull or push the wheelchair when someone is sitting in the wheelchair or it is in manual mode. When there is a suspicious noise, contact your dealer immediately.*


### Protection from electromagnetic interference

The wheelchair should be kept away from strong magnetic fields and large inductive electrical equipment. Pay attention to this if there are sources of electromagnetic interference nearby, so that you are as far away from these sources as possible.


## 3. Adjusting the wheelchair

- Remove the power wheelchair from the packaging.
- Lift the folded backrest and press down on the seat back tubes on both sides until they lock into place. Then, unfold the power wheelchair until the seat cushion and back cushion are fully extended.

*Note: To avoid possible entrapment of limbs, do not hold the seat tubes with your hands when pressing the seat cushion.*

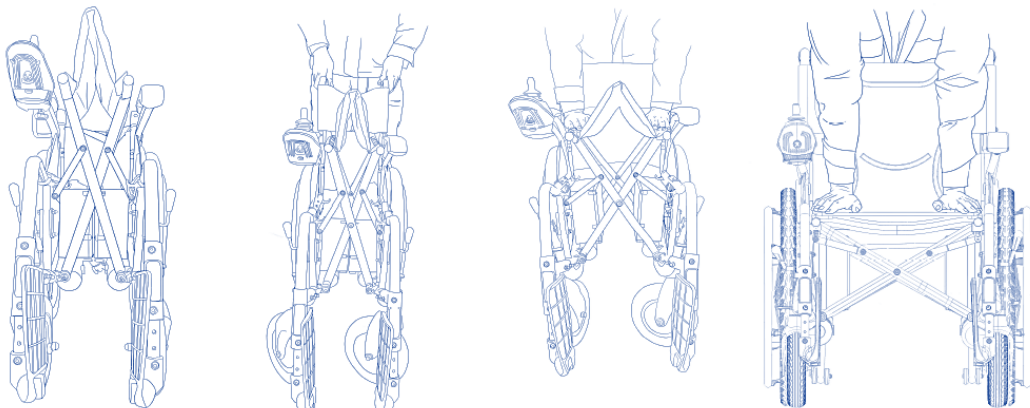
- Using the backrest folding mechanism: hold the handle with both hands, lift the backrest tube until the backrest folding pin locks into its hole. To fold the split backrest, press the folding buckle with your fingers, and hold the handles with both hands. Fold it, making sure that the pin has come out of its hole.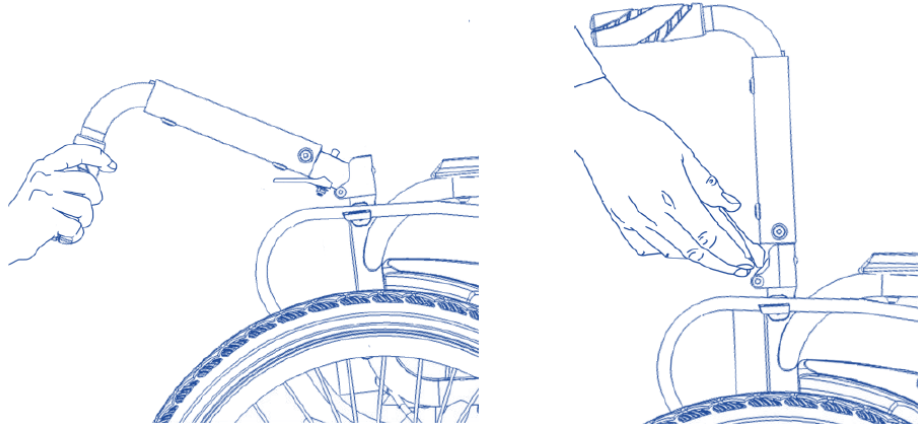


*Note: To avoid possible entrapment of limbs, do not place your fingers in the folding points of the backrest.*

- Footrest height adjustment: loosen the fixing screws with a hexagonal wrench, adjust the footrest to the appropriate height according to the person's height, and secure the fixing screws.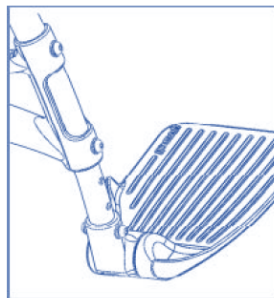


- Installing and removing anti-tip wheels: Hold the anti-tip wheel with your fingers and align its axle with the corresponding hole in the frame. Insert it into the hole until it is fully locked in place on the wheelchair. To remove, reverse the procedure, releasing it from the locking point and carefully removing it from the frame.

*Note: The anti-tilt wheel is approximately 4cm from the ground.*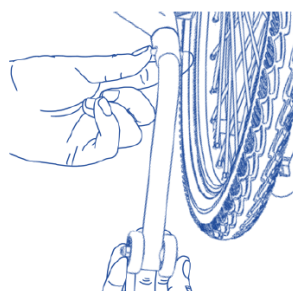

- Adjusting the Controller: While seated in the power wheelchair, loosen the Torx screw securing the controller. Adjust the controller mounting bar by moving it forward or backward to achieve the desired position. Once the controller is properly positioned, retighten the Torx screw to secure it in place.

*Note: The adjustment procedure is the same when the controller is installed on the left side.*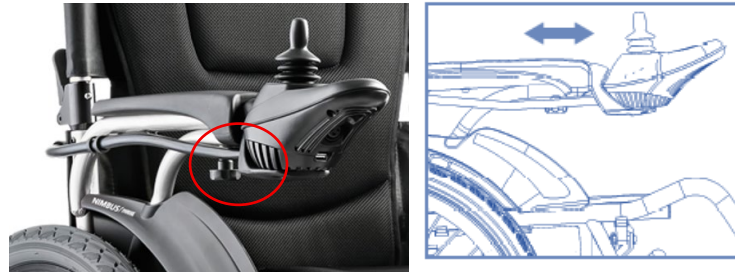


### Switching from manual to electric mode

Manual mode: Turn off and stop the wheelchair, rotate the levers as shown in the following picture “Manual mode”


Electric mode: Rotate the levers as shown in the following picture “Electric mode”

The product has the following sticker for easy identification of the mode switching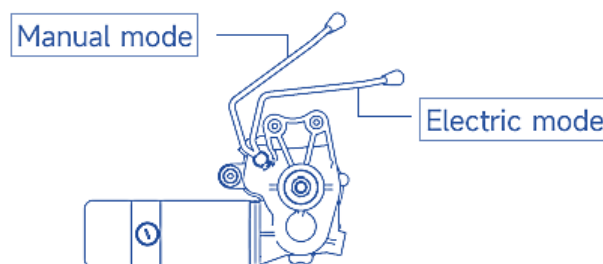

## 4. Battery and charger

The battery is already assembled on the wheelchair. It can be folded and stored without removing the battery.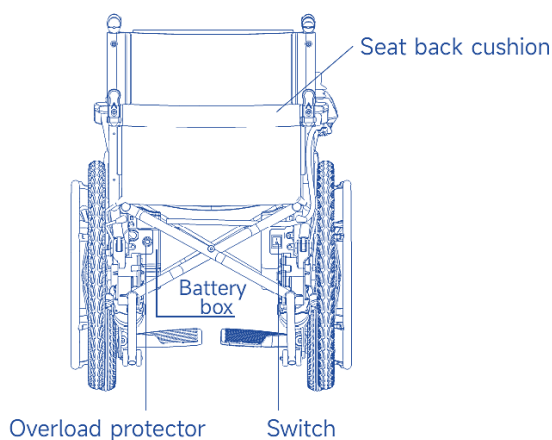


### Using the charger

When charging, turn on the power switch on the battery box of the electric wheelchair and turn off the power button on the controller.


*Note: The power switch on the battery box is off when delivered. Be sure to turn on the power switch when charging. The switch is located on the back of the right battery box:*

"I": ON  
"O": OFF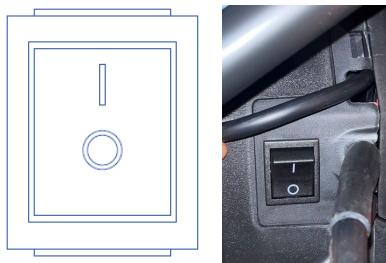

**Do not stop charging until the process is complete. Using the battery without charging it sufficiently will shorten its service life. When the battery is fully charged, the indicator will turn green. After charging is complete, turn off the power, otherwise the battery will discharge slowly. Do not charge for more than 24 hours. Overcharging is dangerous.**

- Insert the charger output plug into the bottom socket of the controller.
- To ensure the correctness of the circuit, do not arbitrarily change the circuit.
- To avoid any trouble, do not disconnect the battery circuit during charging.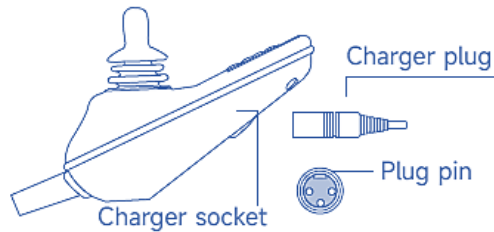

*Users should follow the following rules to avoid hazards when charging:  
Do not expose the wheelchair to sunlight and damp environments when charging.  
The charging ambient temperature range is -20 °C to 50 °C. If it is outside the ambient temperature range, the battery cannot work well and may be damaged.  
It is normal to hear noise during charging as the fan is working to prevent the charger from overheating.  
Prevent liquid from entering the charger when charging. In addition, do not place the charger on flammable objects such as fuel, footrest or seat cushion.*

## **Battery use and maintenance**

Pay attention to the operation indicator on the control panel: if the green indicator is not on, charge the battery as soon as possible. If the red indicator is on, the battery capacity is too low. Charge the battery immediately to prevent the battery voltage from dropping too much, thus affecting its service life.

The battery has obvious positive and negative electrode marks, and a reliable connector is provided to ensure the normal connection of the circuit. Non-qualified persons should not connect the circuit.

The battery is maintenance-free and does not require daily replenishment of supplementary liquid. During charging, the battery temperature will rise, but it should not exceed 45°C. If the temperature exceeds 45°C, stop charging until the temperature drops below 35°C. If the wheelchair is parked for a long time, recharge the battery at least once a month.

Do not use the battery at a temperature of  $\geq 50^{\circ}\text{C}$  or  $\leq -20^{\circ}\text{C}$ .

Keep the battery clean and dry. Do not hit the battery with hard objects. Store it properly and keep it out of reach of children.

The power switch on the battery box cuts off the battery power and reduces the battery's natural energy consumption. Turn off the power switch on the battery box when the wheelchair is not used for a long time.

"Full capacity": develop the habit of keeping the battery at full capacity and charging the battery in time according to use, so that the battery is at "full capacity" in the long term.

## **Battery replacement**


Press the power switch on the battery box to the "O" position to replace the battery.

Only one battery box is provided on the right, and the battery box only has one lithium battery: the red on the battery indicates the positive electrode and the black indicates the negative electrode. The battery must be replaced as follows:


- a. Loosen the 6 screws on the back of the battery box and open the back cover.
- b. Remove the battery from the battery box, separate the connector by hand, install the new battery and replace the connector. The rest of the wiring must remain unchanged.

Waste must be disposed of in accordance with relevant national environmental protection regulations.

## 5. Display & Use of Controller

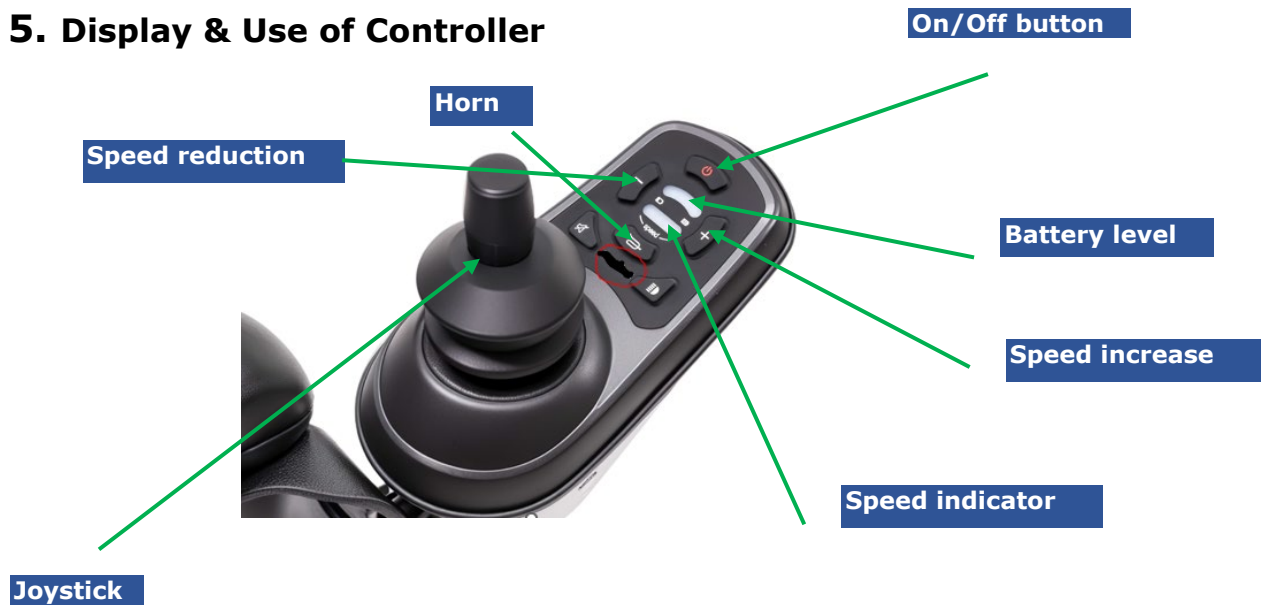<b>Battery level</b>	When the two green LEDs, one red and one yellow are on, it indicates that the battery capacity is full. When the two green LEDs are on, it indicates that the battery capacity is sufficient. When only the yellow or red LEDs are on, it indicates that the battery capacity is insufficient and charge the wheelchair as soon as possible to ensure normal operation.
<b>On/Off button</b>	Press this button to turn the device on or off.
<b>Horn</b>	Press this button to sound the horn.
<b>Speed indicator</b>	Displays the speed setting of the wheelchair. 5 speed settings are available: 1st speed: slowest, 5th speed: fastest.
<b>Speed increase</b>	Press this button to increase speed. The speed increases by 1 each time you press this button and will not change after reaching 5th speed.
<b>Speed reduction</b>	Press this button to decrease speed. The speed decreases by 1 each time you press this button and does not change after reaching 1st speed.
<b>Joystick</b>	The joystick is mainly used to control the direction and speed of the wheelchair. The direction the joystick is pressed is the same as the direction of the wheelchair. The farther the joystick is pressed from the center, the faster the wheelchair will move. The wheelchair will brake automatically when the joystick is released. When the wheelchair is stationary, do not press it forcibly. Hold the joystick with your hand to move the electric wheelchair forward and backward and turn.
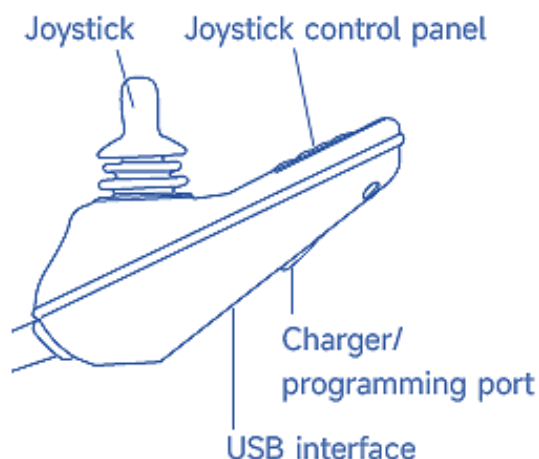
<b>Charger socket/ programming port</b>	Connect the charger to charge the battery.
<b>USB connection point</b>	The USB port provides 5V 0.5A voltage.

## 6. Maintenance & Cleaning

Before maintenance, press the power switch on the battery box to the "O" position. It is highly recommended to have the adjustment and replacement of worn parts by qualified personnel or contact the manufacturer.

INSPECTION INTERVAL	DAILY CHECK	WEEKLY CHECK	MONTHLY CHECK	EVERY 3 MONTHS CHECK	EVERY 6 MONTHS CHECK
BATTERY	✓				
TIRE PRESSURE	✓				
CABLES		✓			
HANDBRAKE			✓		
FRAME				✓	
CONTROLLER				✓	
MOTOR				✓	
CONNECTION				✓	
SEAT -BACKREST CUSHION					✓
TPOXOI					✓
ELECTROMAGNETIC BRAKE					✓


Worn parts are replaced as follows (if difficult to replace, contact the manufacturer immediately for replacement):

- Front wheel replacement: loosen the screws with a wrench, remove the front wheel, install the new front wheel, secure the screws, adjust the screw tightness, and ensure that the front wheel rotates smoothly.
- Rear wheel replacement: ask qualified personnel to replace it or contact the manufacturer.
- Seat-back cushion: loosen the screws with a screwdriver, remove the seat-back cushion, install the new seat-back cushion and secure the screws with a screwdriver.
- Armrest replacement: loosen the screws with a wrench, remove it, attach the new one and secure the screws with a wrench.
- Battery: check the remaining battery capacity. If the battery reaches the end of its service life and needs to be replaced, contact the supplier or manufacturer or purchase a battery of the same specifications locally.
- Tire pressure: It is recommended to inflate tires of 200×45 - 110 and 310×50 - 203 to 260kPa (maximum 325kPa) and inflate tires of 22×1.75 to 280kPa (maximum 345kPa), which can be

adjusted according to the weight and air temperature fluctuation. The tire will gradually leak if stored for a long time or not used, which is normal. Please read the following instructions carefully for operating the wheelchair: a. when the tire pressure is not sufficient, (1) press the tire evenly with your hands, so that the tire and the rim fit evenly. (2) inflate the tire to the appropriate state. b. When the tire is deflated, (1) inflate the tire to about 30% of the full air volume, press the tire evenly with your hands, so that the tire and the rim fit evenly. (2) inflate the tire to the appropriate state.

- Cable: check the electrical parts and connecting cables for damage. If there is any, contact the supplier or have it repaired by qualified personnel. Do not repair it yourself.
- Parking brake: the brake prevents the wheels from moving after the wheelchair is parked and cannot be used when driving. Check whether the brake is damaged. If it is damaged, it can be adjusted and reused.
- Frame: the surface coating of the frame should be wiped with a soft cloth and kept clean. It is forbidden to use lubricant. If the frame is cracked, contact the supplier.
- Controller service: carefully clean the controller and joystick with a cloth dampened with a neutral diluted detergent. Never use abrasive materials or alcoholic detergents for cleaning. Protect the controller from damage when transporting the wheelchair.
- Motor: check for increased oil leakage or noise. If there are any, contact the supplier or manufacturer.
- Maintaining the reliability of the connections: check that the screws and nuts on the body are tightened frequently, and deal with the problems (if any) immediately to ensure driving safety.
- Back-seat cushion: wash the seat cover and backrest with warm water and diluted soapy water. Avoid leaving the wheelchair in a wet place.

## Diagnosis and Troubleshooting

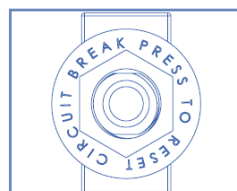If the power signal is not available when the power button on the controller is pressed, confirm whether the power switch on the battery box is in the "I" position. If it is in the "O" position, press the power switch on the battery box to the "I" position. Then, if the power signal is still not available for the power button on the controller, the current may be too large and the overload protection has automatically cut off the

power. At this stage, press the power switch from the "I" position to the "O" position and then back to the "I" position.

In the event of a fault, the controller will sound an alarm and flash, and the corresponding faults can be dealt with according to the times of the alarm sounds.

The diagnostic number displayed by the product's built-in diagnostic function can reflect the nature of the abnormal condition. These abnormal conditions can be detected without other service tools. Audible signal: the first 2 high-frequency sounds are the guide sounds and the following sounds are the alarm sound times in cycles.

DIAGNOSTIC SOUND	DIAGNOSTIC DESCRIPTION	PROPOSED SOLUTIONS
1	LOW VOLTAGE	If the battery voltage is low, use it after charging. If the battery is damaged, replace it. The battery cannot be charged.
2	RIGHT ENGINE FAILURE	Check the right engine for any loose connectors or engine cables.
3	RIGHT BRAKE	Check the right brake for any loose connections or motor cables. Check the brake switch for any damage or poor switch contact.
4	LEFT ENGINE FAILURE	Check the left engine for any loose connectors or engine cables.
5	LEFT BRAKE	Check the left brake for any loose connections or motor cables. Check the brake switch for damage or poor switch contact.
6	OVERCURRENT	Check the motor brake and gearbox for any power outages. If the current detected by the ammeter is not large, the controller may be damaged.
7	ROCKER	The rocker has not been reset or the shuttle valve cable is broken or the connector is loose.
8	CONTROLLER FAILURE	Consult the manufacturer's technical department.

DESCRIPTION OF DAMAGES	CAUSE ANALYSIS AND ELIMINATION METHODS
THE POWER INDICATOR DOES NOT LIGHT UP WHEN THE POWER BUTTON ON THE CONTROL PANEL IS PRESSED	<p>The connection between the battery and the controller is not correct. Reconnect after checking.</p> <p>The battery voltage is too low. If the indicator still does not light up after charging the battery, the battery may have reached the end of its service life. Replace the battery.</p>

	If the controller malfunctions, contact your dealer, manufacturer, or customer service.
LOW BATTERY VOLTAGE	<p>The contact of the connector between the battery and the controller is not reliable. Please re-install it.</p> <p>The contact resistance between the battery connection is large. If the contact resistance is not large, the contact surface may be oxidized or loose. Remove the oxide layer or install the connector correctly.</p> <p>If the battery voltage is too low, it indicates that the battery has reached the end of its service life. Please replace the battery.</p>
HIGH BATTERY VOLTAGE	The battery charging voltage is too high. The voltage should not exceed 29.4V after charging is complete.
THE MOTOR IS NOT WORKING	<p>If the motor fails, contact the dealer, manufacturer or customer service.</p> <p>The motor and controller connector is not connected securely. Please reconnect it.</p>
ENGINE BRAKE FAILURE	<p>The motor connector is not connected securely. Connect it again.</p> <p>The electric brake coil is damaged</p>
THERE IS NO CHARGING INDICATION	<p>The motor and controller connector is not connected securely. Reconnect securely.</p> <p>The battery has reached the end of its service life or the charger is damaged. Replace the battery or charger.</p>
DRIVING DISTANCE DECREASES AFTER CHARGING	<p>The battery is not fully charged. Please charge it again.</p> <p>The battery is nearing the end of its service life. Please replace it.</p>

## 7. Electromagnetic compatibility

This product complies with the requirements of the relevant content of the EMC (electromagnetic compatibility) standards IEC60601-1-2 and ISO7176-21.

The user must assemble and operate the product in accordance with the instructions for use attached to the electric wheelchair.

Portable equipment and radio frequency (RF) communication equipment may affect this product. To avoid strong electromagnetic interference, do not use this product near a mobile phone, microwave oven, etc.

Do not use this product adjacent to or stacked with other equipment. If it is necessary to use the product adjacent to or stacked with other equipment, observe and ensure that the product operates normally under the circumstances.

Guidance and declaration of manufacturer -Electromagnetic emission	
Immunity test	Compliance
Electrostatic discharge (ESD) IEC 61000-4-2 ISO7176-21	±8 kV contact ±15 kV Air
Electrical fast transient/bursts IEC 61000-4-4 ISO7176-21	±2 kV for power supply lines
Surge IEC 61000-4-5 ISO7176-21	±1 kV line to line
Voltage dips, short interruptions and Voltage variations on power supply input lines IEC 61000-4-11 ISO7176-21	0% U <sub>r</sub> 0.5 cycle at 0°, 45°, 90°,135°, 180°, 225°, 270° and 315°
	0% U <sub>r</sub> 1 cycle 70% U <sub>r</sub> 25/30 cycles at 0°
	Voltage short interruptions : 0% UT 250/300 cycles at 0°
Power frequency (50 Hz) magnetic IEC 61000-4-8 ISO7176-21	30 A/m
Radiated RF EM fields IEC 61000-4-3 ISO7176-21	20V/m 80MHz - 2.7GHz 80% AM at 1kHz
Conducted disturbances induced by RF fields IEC 61000-4-6 ISO7176-21	3V/m 0.15MHz - 80MHz 6V in ISM and amateur radio bands between 0.15MHz and 80MHz 80% AM at 1kHz
Note: U <sub>r</sub> is the a.c. mains voltage prior to application of the test level.	

#### Electric and magnetic environment guidance in use

Guidance and declaration of manufacturer-Electromagnetic emission	
Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Emission of harmonics IEC 61000-3-2	NA
Voltage fluctuations / flicker emissions IEC 61000-3-3	NA

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment						
Test frequency (MHz)	Band (MHz)	Service	Service	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 - 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27
450	430 - 470	GMRS 460, FRS 460	FM ±5 kHz deviation 1 kHz sine	2	0,3	28
710	704 - 787	LTE Band 13,17	Pulse modulation 217 Hz	0,2	0,3	9
745						
780						
810	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0,3	28
870						
930						
1720	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1,3,4,25; UMTS	Pulse modulation 217 Hz	2	0,3	28
1845						
1970						
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28

5240	5100 - 5800	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0,3	28
5500						
5785						
<p>NOTE If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.</p>						

## **8. Transport and storage**

### **Transport**

Handle the product carefully during transportation. Do not drop, turn over or apply excessive pressure. Products can be stacked in a maximum of 2 layers.

### **Storage**

This product should be placed in a dry and ventilated place and should not be stored in a place subject to high temperatures and sudden temperature changes. This product should be separated from acids, alkalis and other chemically corrosive objects.

### **Environmental restrictions for transportation and storage**

Ambient temperature range: -40°C~+65°C

Relative humidity range: 10%~100%, non-condensing

Atmospheric pressure range: 86kPa~106kPa

## **9. Incident reporting**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides

## **10. Declaration of conformity**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

## 11. Warranty

The product carries a three\* (3) years warranty for the frame from the date of purchase.

\*One (1) year for the lithium battery, controller and motor.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time like brakes, wheels and plastic parts.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems, carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer

### Warranty Claim Form

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT:			
DEALER DETAILS:			
FULL NAME:			
DDRESS:			
PHONE:			
			SIGN AND SIGNATURE

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Description of symbols that have been printed on the label, Master Carton and/or User Manual

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MOBIAK S.A