GR EN

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INTENDED USE:

The Anti-Decubitus Cushions have been designed to reduce risk of decubitus. They can be used for disabled and elderly people who maintain a wheelchair and sedentary lifestyle for a long time.

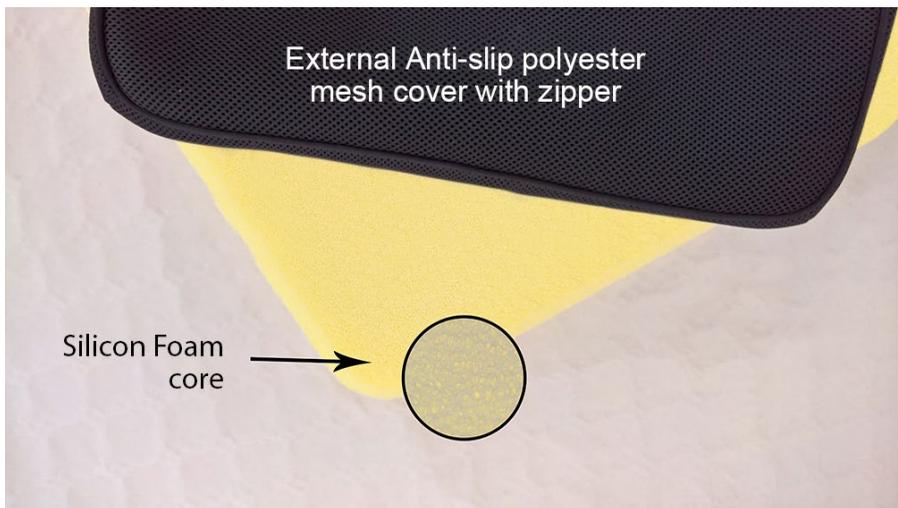
It can also be used for users who do not have any symptoms and want to offer more comfort in their daily life. The Anti-Decubitus Cushion should only be used by users that have some mobility.

The Anti-Decubitus Cushion can be used by the user himself.

While sitting on the cushion, it is important that the user gets up or is lifted up as many times as possible to limit sustained compression of pelvis in the vicinity of protruding bones.

FEATURES:

- Ergonomic design
- It offers high comfort and stability to the user
- Lightweight and easy to carry
- Breathable material
- It offers reduction of humidity and heat build-up
- External cover with zipper, removable and washable
- Antimicrobial properties
- Fire retardant properties

CLEANING & WASHING INSTRUCTIONS

CUSHION

- The cushion is not machine washable but can be hand washed and disinfected with water up to 95°C for optimal hygiene.
- The foam interior of the pillow may take some time to dry naturally after this process.
- Do not use detergents or other solvents.
- Do not wash it in the washing machine.
- Do not use a steam cleaner.
- Do not use bleach
- Do not ironing.
- Do not use dry cleaning.

COVER

- The external cushion cover is removable and machine washable at 71°C.
- The zipper must be closed when the cover is washing in the washing machine.
- Do not use a steam cleaner.
- Do not use bleach.
- Do not ironing.
- Do not use dry cleaning.

TECHNICAL SPECIFICATIONS:

MBK CODE	DESCRIPTION	WITH	DIMENSION	MAX LOAD
0803124	CUSHION "SILICONE I" (50x45x5.5)	INTERNAL SILICONE FOAM CORE	50 x 45 x 5.5cm	120 Kg
0803125	CUSHION "SILICONE II" (45x40x5.5)	INTERNAL SILICONE FOAM CORE	45 x 40 x 5.5cm	120 Kg

CAUTION(!)

- Do not smoke & do not use any flame near the cushion
- Do not expose the cushion to direct heat.
- Clean the product in weekly frequency.
- Protect the parts from scratches, cuts and punctures.
- Always check that the product is in good condition with no signs of wear.
- In case of damage or malfunction, please contact your authorized and trained dealer.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product or use other than that mentioned in this manual.
- MOBIAK S.A reserves the right to make changes on the device and the the information contained in this document without previous notice.

STORAGE CONDITIONS

The cushion must be stored in a dry and clean environment at a temperature between 0 - 40°C.

PROTECTING THE ENVIRONMENT

If your product needs replacement, or no longer serves you, consider the environment.

- 1) Do not dispose your product along with the municipal waste (this is also the importance of the shown recycling symbol).
- 2) Contact your municipal authority to indicate the point of disposal of your product for recycling
- 3) You have to dispose your useless product in the right place.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

NORTON SCALE DETAILS

The Norton Scale was developed in the 1960s and is widely used to assess the risk for pressure ulcer in adult patients. The five subscale scores of the Norton Scale are added together for a total score that ranges from 5-20. A lower Norton score indicates higher levels of risk for pressure ulcer development.

SCORE	RISK FOR PRESSURE ULCER DEVELOPMENT
>18	LOW
18-14	MEDIUM
14-10	HIGH
< 10	VERY HIGH

Please be noted that the Norton Scale is a tool only to be used by trained healthcare professionals, and only as a supplement to a thorough patient state evaluation.

MBK CODE	EN DESCRIPTION	SUITABLE FOR NORTON SCALE SCORE
0803124	CUSHION "SILICONE I" (50x45x5.5)	Between 14 and 10 High Risk
0803125	CUSHION "SILICONE II" (45x40x5.5)	Between 14 and 10 High Risk

Declaration of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

WARRANTY:

The product has a warranty of good operation for TWO (2)* years from purchase date.

The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product instructions for use.

It also does NOT cover worn parts and whatever it is going to be damaged during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where provided), & transport companies are also not covered by the warranty.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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