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INTENDED USE:

The WAVE bath chair is intended to provide support to children/ teenagers during bathing or showering. The bath chair is intended for indoor use in a bathroom environment. It provides excellent support and stability during bathing, allowing caregivers to safely and comfortably bathe children/ teenagers who may have difficulty sitting up or maintaining their balance.

SPECIFICATIONS & DIMENSIONS: Please refer on **ANNEX A**

ASSEMBLY: Please refer on **ANNEX B**

SYMBOLS: Please refer on **ANNEX C**

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Do not exceed the maximum loading capacity.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts.
- Use extra caution when operating the bath chair in a bathtub.
- Never leave the user unattended. The carer shall always be in contact with the user.
- Never let the user stand up in the bath chair.
- Do not place the bath chair on uneven or sloping ground.
- All the accessories must be carefully adjusted for the comfort and security of the user.
- The bath chair should be inspected regularly for signs of damage and wear. Do not use the bath chair if any part is broken, torn or missing. The damaged or worn parts must be repaired immediately.

BEFORE USE:

- Check whether the frame is damaged in order to ensure the safe use of the product. There are no cracks or breaks in the frame
- Check whether the product is assembled correctly.
- Check that the brakes are working properly.
- Always check the state of wear of the mechanical parts in order to ensure the absolute safe use of the product for people and objects
- With the exception of the calf support tilt adjustment, the bath chair must not be adjusted while in use.

MAINTENANCE:

- This product should be inspected, maintained, cleaned and/or disinfected regularly.
- Do not use this bath chair if it is damaged, broken or disassembled.
- Contact your supplier for qualified repair or replacement. Do not modify or repair by non-qualified personnel.
- After using the bath chair, it is important to rinse with water to remove any soap or other bath product residue.
- Allow the bath chair to dry in an open position and fold it up after it is completely dry for storage.
- If the environment is humid, store the bath chair and its accessories in a relatively dry place and allow them to dry in an open position to prevent mold growth.
- Continuous and prolonged exposure to sunlight may change the color of the product materials.

CLEANING:

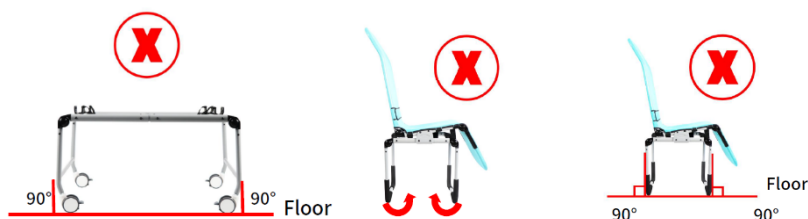
In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain the appearance of the product, the upholstery should be cleaned regularly.
- The frame of the bath chair can be cleaned with a damp cloth and a mild detergent.
- Rinse with clean water before allowing it to dry.
- DO NOT leave the product wet.
- The upholstery of the bath chair is removable and can be machine washed at 40°C/100°F using a mild detergent.
- Tumble dry on a low setting to dry the upholstery.
- Place the bath chair upholstery in a laundry bag to avoid damage to the upholstery and to the washer/dryer.
- The frame should be cleaned regularly with a mild detergent.
- If the product needs to be disinfected, use a common, mild disinfectant.
- Protect the product from abrasions, cuts and punctures.
- The bath chair accessories are hand washable at 30°C / 85°F with a mild detergent. Do not tumble dry the accessories.
- You can reattach them to the bath chair after allowing them to dry completely naturally.
- Do not use bleach.
- Do not iron.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAC S.A reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$ Deviation for the Dimensions of the Products.
- If the wheeled base or base support of the product are positioned at 90 degrees, there is a high risk of it tipping over.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

WARRANTY:

The product is guaranteed for a period of three (3) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels or brakes.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE::	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
			SIGN AND SIGNATURE

 **MOBIAM S.A**
KAOIANA AKPOTHPIOY, XANIA - KPTH
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

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Adjusting the backrest:

Lift the D-ring on the back of the backrest to adjust the backrest angle.

Stop pulling the D-ring once the backrest is at the desired angle, and ensure the bar is inserted into the hole in the connection part between the backrest and seat.

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Adjusting the calf support:

Press and hold both buttons.

Rotate the shin support to the desired position, release both buttons until they click into place.

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Adjusting the rear support legs:

Press and hold both buttons.

Rotate the rear support legs to the desired position, release both buttons until they click into place.

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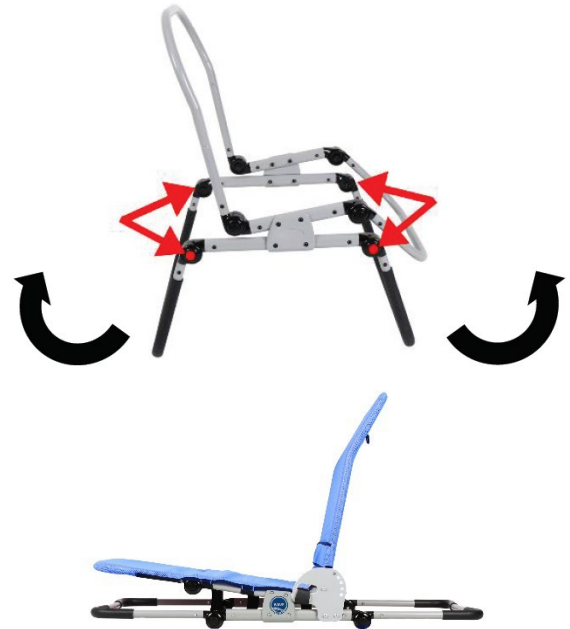
Adjusting the rear & front support leg section:

Press and hold both buttons.

Rotate the rear and front support sections to the desired position, release both buttons until they click into place.

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
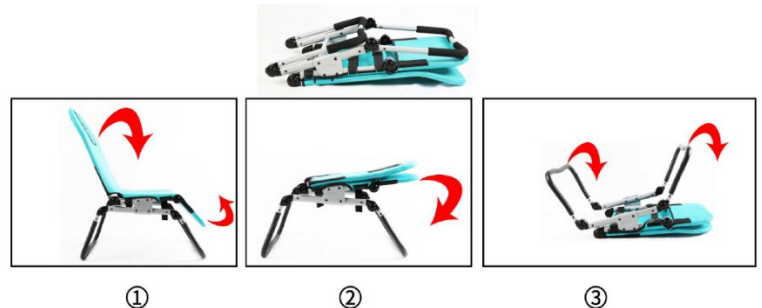
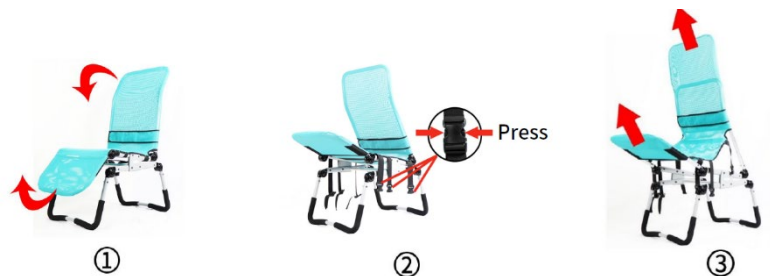
Assembling the wheeled base

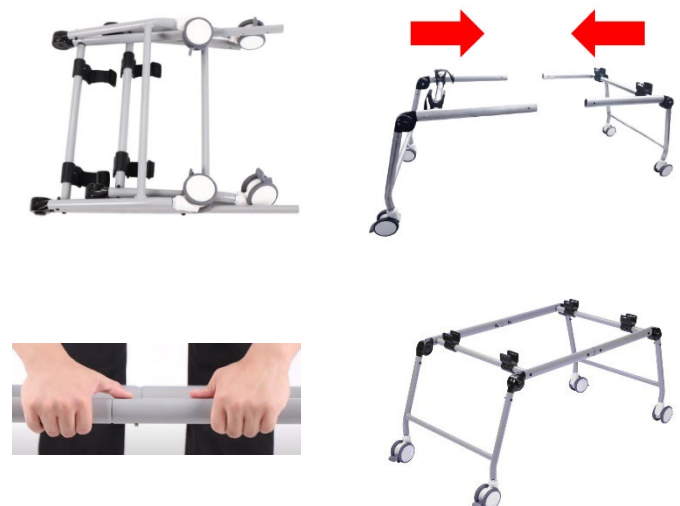Please refer to the following related images. To assemble, press and hold the button on the slender tube, then insert it into the larger tube until the button pops out of the hole in the thick tube. Once you've ensured the stand is securely locked and stable, it is ready for use.

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
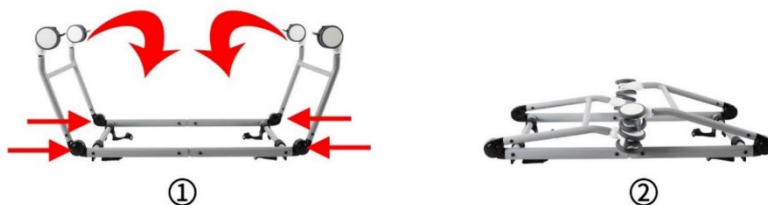
Placing the bath chair on the wheeled base.

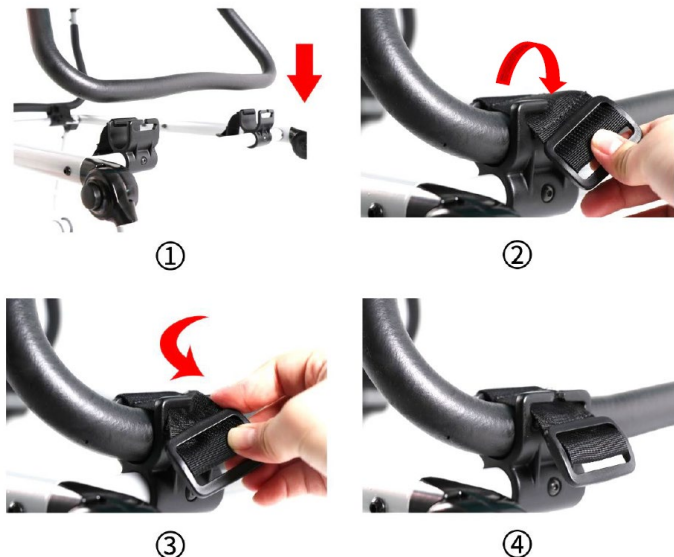Please refer to the following related images.

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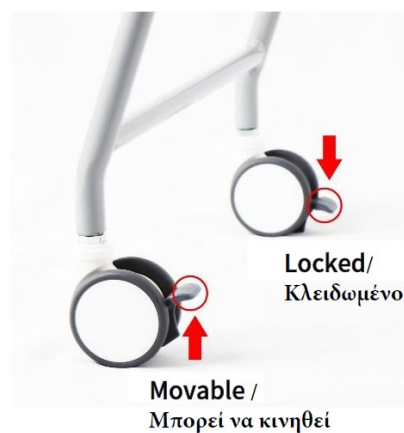
Castors of wheeled base

Please refer to the following related images.

To avoid falls or injuries, always lock all wheels when placing or removing the user.

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Attaching the accessories to the bath chair

Please refer to the following related images.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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