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INTENDED USAGE:

The foldable transfer wheelchair is designed for the transport of people with mobility problems or limited mobility and can be used with the help of an operator. The lightweight frame of the transport wheelchair is made of aluminum, which makes it easy to transport and use. It offers stability with safety belt, a portable design with an ergonomic carrying handle and easy folding. It is user-friendly as it does not require assembly and is ready to use.

INDICATIONS AND CONTRAINDICATIONS:

The foldable transfer wheelchair is designed for transporting people with mobility problems or limited mobility and can be used with the assistance of an operator. The user must be able to maintain balance on their own when sitting and when lifting as well as have sufficient ability to maintain themselves in a sitting position. It should not be used if the user has problems related to balance or perception disorders.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- If you don't read this user manual, it is preferable not to use this product or another available part.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a ± 3%. Deviation for the Dimensions of the Products.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts
- Protect the product from scratches, cuts and punctures.

SAFETY WARNINGS AND INDICATIONS:

- The maximum user weight indicated in the instructions must not be exceed.
- Before every use, be sure is correctly assembled in a stable position.
- Can be used only on surface that are free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting or assembling process.
- Do not use the product if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- Do not use the product for a different purpose from the intended one.
- The product must not be stored outdoors.
- Keep the product away from heat sources.
- Always check the state of wear of the mechanical parts to ensure the completely safe use of the product for people and objects

MAINTENANCE INFO:

In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user must be certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

- Gloves should be worn.
- Remove the evident dirt before to proceed with the most accurate disinfection.
- Use water or mild detergent for the cleaning procedure.
- Wipe with a dry and clean cloth.
- If you need to disinfect the product, use a common, mild disinfectant

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the referred standards and legislation.

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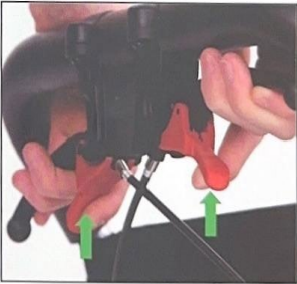
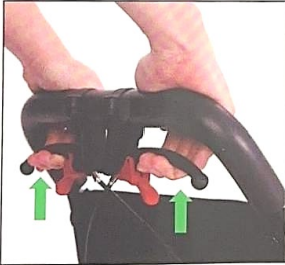
WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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