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INTENDED USE:

The product is used for the transportation of patients in emergency or in situations demanding a quick and safe evacuation of the patient. It is also used for horizontal transport, lifting and immobilization of a patient with a suspected spinal injury. It has to be used exclusive by professional and qualified personnel, duly trained for emergency care.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- There is a ± 3% Deviation for the Dimensions of the Products.
- It is always recommended to follow the procedures designed for emergency response.
- Improper use of the device can cause injury to the patient.
- Improper use of the device can cause injury to the patient.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this device or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the device because may be caused damage.
- Carefully READ the entire manual before using the product.
- Save this document for future reference.
- The product should be used only by qualified personnel.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play or operate with the device.
- Special care is always required where there are moving parts that can cause limb entrapment or injury.

PRODUCT PRESENTATION:

- The stretcher is made of high-density polyethylene – HDPE. This makes it rigid, lightweight and reliable for use in special situations.
- It has 18 handles around the perimeter for operator convenience.
- Its material is durable with high strength and non-absorbent. It is also easy to clean and disinfect.
- The maximum Load is 170Kg.
- It includes four (4) safety belts, quick-release, with plastic buckle and 5 cm width for the restraint of the injured person.

- It can be used on boats, ambulances, hospitals, in sports and mountaineering clubs and generally outdoors for the transport of patients and injured people.
- It can float on water.
- It can be used in combination with the Head Immobilization Splint 0805908.
- It can be used in combination with the inflatable Set (leg and hand) splints with set code 0805909.
- It is compatible for use with X-ray and MRI devices. (X-Ray & MRI compatible)

TECHNICAL CHARACTERISTICS

The package contains: main product x1, seat belt x 4 & user manual x1.

REFERENCE NUMBER	0805907
NAME	SPINE BOARD
FRAME	ONE PART
FRAME MATERIAL	HIGH DENSITY POLYETHYLENE – HDPE
COLOR	ORANGE
PRODUCT DIMENSIONS	185X45X6 CM
MAXIMUM LOAD	170 KG
PRODUCT NET WEIGHT	7 KG
SAFETY BELTS	4 PIECES, QUICK-RELEASE WITH PLASTIC BUCKLE, 5CM WIDTH.
HANDLES	18 PERIPHERAL HANDLES
OTHER	ABILITY TO FLOAT IN WATER WITH A LOAD UP TO 80KG
X-RAY & MRI COMPATIBILITY	COMPATIBLE
WAY OF TRANSFER	HORIZONTAL
MASTER CARTON DIMENSION / GROSS WEIGHT	185X47X7 CM / 1 PC PER MASTER CARTON / 8,80 KG

BEFORE EVERY USE:

- Check if the frame is damaged in order to guarantee a safe use of the product (there are no cracks or fractures in the frame).
- Check that the surface of the stretcher is not cracked.
- In case of damage do not use the product and contact your dealer for further instructions.
- Always check the condition of wear for all the parts to ensure the absolute safe use of the product for people and objects.
- Periodically check its sturdiness and the stability.
- Check that the safety belts for patient restraint and immobilization are present, intact and functional.

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USE OF PRODUCT:

1. In case you need to use the head immobilizer, or a collar, put it on the product first.
2. Successfully move the patient or user onto the stretcher and secure with the belts.
3. Use the Handles to secure the safety belts on the stretcher.
4. The presence of 18 handles around the perimeter allows you to lift the patient by grasping the stretcher from various sides. Avoid using a handle that you have passed the seat belt.
5. Before lifting the stretcher, make sure that you have properly strapped the patient with the safety belts.
6. DO NOT use with patients who exceed the weight limit on the product label and in this user manual.
7. Only manual lifting and at least four operators are allowed.
8. Operators must be positioned symmetrically and in such a way that the whole patient can be lifted safely and steadily. If four operators are not sufficient to ensure the safety of the patient, then two additional handlers shall be placed on either side of the center.
9. Always inform the patient when adjusting to the stretcher. Never leave stretcher unattended with patient inside.
10. The device maybe has consequences for patients related to prolonged laying on a rigid, non-anatomical surface. So, the actual need for use of this stretcher must therefore be carefully assessed by the rescuer in accordance with his/her training guidelines.

CAUTION: For safe use in water, the operators should be at a depth where they can stabilize their feet on the bottom, because the foaling of the stretcher depends on the weight of the patient.

Requirements for operators:

- To be trained to deal with corresponding situations.
- Or have practical knowledge of handling patients in emergency situations.
- Be familiar with the procedures and the stretcher.
- Be physically able to assist a patient.

MAINTENANCE:

Check the stretcher after each use. The inspection is intended to investigate anything that may compromise the integrity and proper operation of the stretcher. Check the product for any looseness, cracks, breaks, bent or damaged parts.

CLEANING AND DISINFECTION:

- Clean the plastic parts with water, rinse and wipe well with a soft and clean cloth and dry away from direct sunlight.
- You can use a wet cloth and mild soap on the surfaces that need a better cleaning.
- DO NOT leave the product wet or soiled with dust or mud.
- Do not use chemical cleaners to clean the frame. This may damage the product's coating surface.
- CAUTION: Do not use acidic or caustic solutions. Do not use solvents or stain removers.
- If you must disinfect the product, use a common, mild disinfectant, after reading the instructions provided by the disinfectant manufacturer.
- Protect the product from scratches, cuts and punctures.

CONDITIONS OF STORAGE AND TRANSPORT

The product should be stored packaged under the following conditions:

- For long-term storage the product should be placed in a dry and ventilated place and should not be placed in an environment with high and rapid changes of temperature.
- DO NOT store the product in locations that are close to high temperature sources or direct sunlight for long periods of time.
- The product should be isolated from chemicals and corrosive substances.
- The product should be packaged to protect it from dust, and parts that may be affected or damaged by oxidation should be lightly lubricated.
- During transportation and storage, when packed, the maximum stack height is six layers.
- The product can be transported by any common vehicle.
- Temperature range: -20°C ~ 60°C.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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