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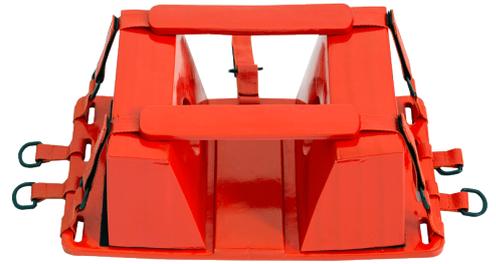
1. Introduction

We would like to thank you for choosing a MOBIAK SA product. The product user manual is an important document that provides you with information on how to use it and technical features. Please read the user manual carefully before using the product. Learn the product and practice before any application. This manual applies to our model: **0805908**.

- The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design. Please check it carefully.
- The product you have in your hands may differ from the one in the following photos. The assembly process is similar.

SYMBOLS: Please refer on ANNEX A

2. Intended Use:

Head Immobilizer is used to help the fixation of the head and neck of the injured people. It has to be used exclusive by professional and qualified personnel, duly trained for emergency care.

3. CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- It is always recommended to follow the procedures designed for emergency response.
- Improper use of the device can cause injury to the patient.

4. GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this device or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the device because may be caused damage.
- Carefully READ the entire manual before using the product.
- Save this document for future reference.
- The product should be used only by qualified personnel.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play or operate with the device.
- Special care is always required where there are moving parts that can cause limb entrapment or injury.

5. Product Presentation:

- It consists of two sections made of foam material with vinyl lining, which are fixed to a base with straps so that it can be adapted to different types of stretchers.
- The Head Immobilizer 0805908 can be used with the Spine Board 0805907 and the Scoop Stretcher 0805910.
- The head strap is designed to conform patient's forehead.
- There are holes for monitoring the patient's ear.
- it is designed to support the injured person's head naturally, avoiding as much as possible extra pressure on the head.
- Minimal interference with X-ray, MRI, or CT procedures.
- Waterproof plastic coating makes it easy to prevents bacterial growth within components. Will not absorb blood or bodily fluids. Durable and easy to clean.
- Direct contact with skin, can be used in very cold weather.
- It can be used on boats, ambulances, hospitals, in sports and mountaineering clubs and generally outdoors for the transport of patients and injured people.

6. Package Contents

The package contains: main product x1, fixing belts for stretcher x2 & user manual x1.

7. Specifications

Reference Number	0805908
Name	HEAD IMMOBILIZER ADULT
Frame Material	High Density Foam with Vinyl lining
Product Parts	Base Board with eight "D" shape loop x1, Block with round hole x 2, Immobilization Belt x2, Head Strap x2
Color	Red
Product Dimensions	43,5x27x14 cm
Product Net Weight	1 Kg
X-RAY & MRI Compatibility	COMPATIBLE
Temperature Range	-20°C ~ 50°C
Master Carton Dimension / Gross Weight	56x45x60 cm / 8 pcs per Master Carton / 10 Kg

8. Before Each Use:

1. The actual need for use of this stretcher must therefore be carefully assessed by the rescuer in accordance with his/her training guidelines.
2. Check if the frame is damaged in order to guarantee a safe use of the product (there are no cracks or fractures in the frame).
3. Check that the surface of the product is not cracked.
4. In case of damage do not use the product and contact your dealer for further instructions.
5. Always check the condition of wear for all the parts to ensure the absolute safe use of the product for people and objects.
6. Periodically check its sturdiness and the stability.
7. Check that the safety belts for patient restraint and immobilization are present, intact and functional.

9. Use of the Product:

1. **Central top belt:** Put base board on spine board, insert the top belt through the central hole on spine board (Picture 1).
2. Have belt go through the D loop, reverse and stick on belt velcro (Picture 2).

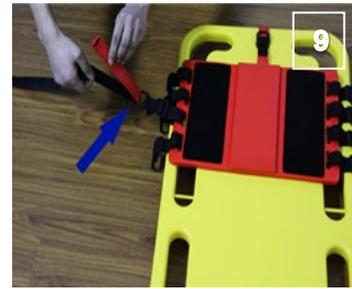
3. **Two belt and the Middle four D loop to immobilize base board on spine board:** Immobilize one side velcro to D loop (Picture 3, 4)

4. Then turn over the spine board and immobilize the other side velcro to symmetrical D loop. (Picture 5, 6)

5. **Two Block to immobilize patient's head:** Firstly immobilize a Block on base board's one side by velcro (Picture 7).
 6. Put patient's head carefully on the base board (Picture 8).
 7. Put the other Block to the other side of head and get the proper position for head immobilization, stick to velcro. Neither leave a too large space between two blocks or immobilize head uncomfortably too tight. Hole on block is used to monitor patient ear (Picture 9).

8. **Put on two Head Strap:** Use one strap and two symmetrical D loop to immobilize forehead (Picture 10).
 9. Use the other strap and left two symmetrical D loop to immobilize the chin (Picture 11)

10. **Requirements for operators:**

- To be trained to deal with corresponding situations.
- Or have practical knowledge of handling patients in emergency situations.
- Be familiar with the procedures and the stretcher.
- Be physically able to assist a patient.

11. **Maintenance:**

Check the product after each use. The inspection is intended to investigate anything that may compromise the integrity and proper operation of the stretcher. Check the product for any looseness, cracks, breaks, bent or damaged parts.

WARNING

In the event of a problem occurs, **DO NOT** use the product until the issue will be repaired.

12. **Cleaning And Disinfection:**

- Clean the plastic parts with water, rinse and wipe well with a soft and clean cloth and dry away from direct sunlight.
- You can use a wet cloth and mild soap on the surfaces that need a better cleaning.
- Don't wash black Velcro parts with water.

- DO NOT leave the product wet or soiled with dust or mud.
- Do not use chemical cleaners to clean the frame. This may damage the product's coating surface.
- CAUTION: Do not use acidic or caustic solutions. Do not use solvents or stain removers.
- If you must disinfect the product, use a common, mild disinfectant, after reading the instructions provided by the disinfectant manufacturer.
- Protect the product from scratches, cuts and punctures.

13. Conditions of Storage and Transport

The product should be stored packaged under the following conditions:

- For long-term storage the product should be placed in a dry and ventilated place and should not be placed in an environment with high and rapid changes of temperature.
- DO NOT store the product in locations that are close to high temperature sources or direct sunlight for long periods of time.
- The product should be isolated from chemicals and corrosive substances.
- The product should be packaged to protect it from dust,
- The product can be transported by any common vehicle.
- Temperature range: -20°C ~ 50°C.

14. Environmental Protection Statement:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

15. Incident Reporting:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

16. Declaration Of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

17. Terms of Warranty:

The product is guaranteed for a period of two (2) years warranty on the frame from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, wheel tire, safety belt, footrests or brakes.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

When the user or the provider apply on warranty request, provide a valid proof of purchase and the completed warranty form, with a photo of the product serial number and a photo or video of the problem that has occurred. In the event the user cannot provide the above documents, if we can confirm that the product or part is still within the warranty period, we may be able to honor the user's request for coverage under warranty, but we are not obligated to do so.

18. Warranty Form

BUYERS DETAILS:	
FULL NAME:	
PHONE:	
PURCHASING DATE:	
INVOICE NUMBER:	
SERIAL NUMBER:	
LOT:	
DEALER DETAILS:	
FULL NAME:	
ADDRESS:	
PHONE:	
DEALER SIGN AND SIGNATURE	

19. Disclaimer:

The content of the product manual is the intellectual property of MOBIAK S.A. (hereinafter referred to as the "Company") and the copyright is reserved by the Company. Without the written consent of the Company, no one may copy, export or translate the manual or any content included in any form into other languages without the written consent of the Company. The Company carefully compiles the product manual with a sense of responsibility for users but does not guarantee the completeness or correctness of its contents. The product manual, as a purely technical document, does not include implied content or content with oblique reference to third parties. The Company assumes no responsibility for any typographical errors that may cause ambiguities. In the event of direct or indirect loss of information or business interruption due to the product manual or the product information contained therein, the Company shall not be liable.

When using all MOBIAK S.A. products, users must strictly follow the product manual and the instructions for use available on the Company's official website (www.mobiakcare.com) for the handling of the equipment and not violate any warning marked in the product manual when using the equipment. In case of injury to users or other persons, or material damage caused by the handling of the equipment without following the instructions for use or by violating the warnings marked in the product manual, the persons concerned shall bear the respective legal responsibilities.

As the Company continues to improve and update its products, it is essential that you regularly log on to its official website to refer to the most up-to-date instructions and conditions of use or contact your distributor for the latest user manual. The Company reserves the right to revise the product manual without notice.

The Company reserves the right of final interpretation.

MOBIAK S.A
 ΚΑΘΙΑΝΑ ΑΚΡΟΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
 KATHIANA AKROTIRIOU CHANIA - CRETE
 T: +30 28210 63222 | WEB: www.mobiakcare.com

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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