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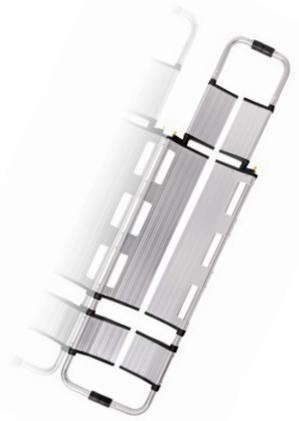
1. Introduction

We would like to thank you for choosing a MOBIAK SA stretcher. The product user manual is an important document that provides you with information on how to use it and technical features. Please read the user manual carefully before using the product. Learn the product and practice before any application. This manual applies to our model: **0805910**.

- The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design. Please check it carefully.
- The product you have in your hands may differ from the one in the following photos. The assembly process is similar.

SYMBOLS: Please refer on ANNEX A

2. Intended Use:

The product is used for the transportation of patients in emergency or in situations demanding a quick and safe evacuation of the patient. It is also used for horizontal transport and lifting to "scoop" patients from the ground or floor, in confined spaces or when the patient needs to be moved with a slight change in body position. It has to be used exclusive by professional and qualified personnel, duly trained for emergency care.

3. CAUTION (!):

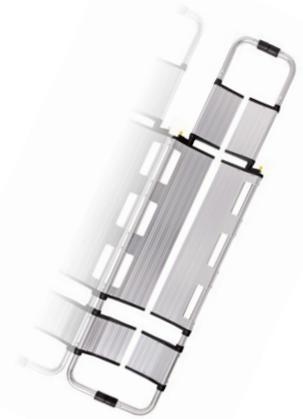
- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- It is always recommended to follow the procedures designed for emergency response.
- Improper use of the device can cause injury to the patient.

4. GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this device or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the device because may be caused damage.
- Carefully READ the entire manual before using the product.
- Save this document for future reference.
- The product should be used only by qualified personnel.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play or operate with the device.
- Special care is always required where there are moving parts that can cause limb entrapment or injury.

5. Product Presentation:

- The stretcher is made of anodized aluminum with aluminum locking mechanisms, which makes it **lightweight, durable, flexible, and reliable** for use in special situations.
- Its material is with high strength and non-absorbent. It is also easy to clean and disinfect.
- It is foldable, splits into two parts so that it can be stored and transported easily.
- It has 6 handles around the perimeter for operator convenience, but both the front and rear can be used as handles.
- The stretcher frame is made in two halves that can be detached and attached under the patient without requiring the patient to be moved.
- The length is adjustable in four positions from 168 cm to 200 cm and can be adapted to most body types.
- The maximum Load is 170Kg.
- It Includes four (4) safety belts, quick-release, with plastic buckle and 5 cm width for the restraint of the injured person

- It can be used on boats, ambulances, hospitals, in sports and mountaineering clubs and generally outdoors for the transport of patients and injured people.
- It is not compatible for use with X-ray and MRI devices. (X-Ray & MRI non-compatible)


6. Package Contents

The package contains: main product, x1, seat belt x 3 & user manual x1.

7. Specifications

Reference Number	0805910
Name	Scoop Stretcher
Frame	Folding in two parts, for easy transport in vehicles
Frame Material	Anodized aluminum with aluminum locking mechanisms
Product Dimensions	168-200x44x7 cm
Product Dimensions Folded	120x44x9 cm
Length of predetermined locking positions	#0:168cm / #1: 178cm / #2: 189cm / #3: 200cm
Maximun Load	170 Kg
Product Net Weight	7,4 Kg
Safety Belts	3 pieces, quick-release with plastic buckle, 5cm width.
Handles	6 peripheral handles & The front and rear part can be used as handles.
X-RAY & MRI Compatibility	NOT COMPATIBLE
Way of Transfer	Horizontal.
Master Carton Dimension / Gross Weight	170x45x9 cm / 1 pc per Master Carton / 10Kg

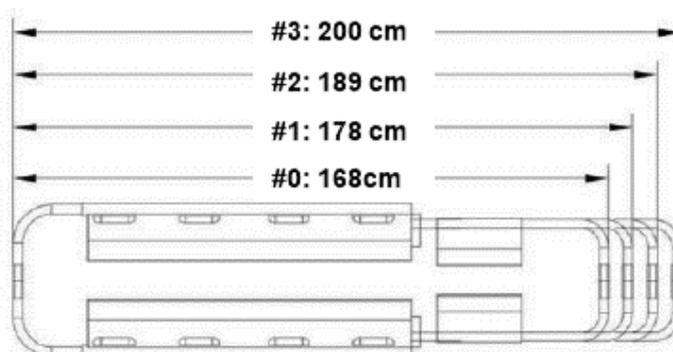
8. Before Each Use:

1. The actual need for use of this stretcher must therefore be carefully assessed by the rescuer in accordance with his/her training guidelines.
2. Check if the frame is damaged in order to guarantee a safe use of the product (there are no cracks or fractures in the frame).
3. Check that the surface of the stretcher is not damaged.
4. In case of damage do not use the product and contact your dealer for further instructions.
5. Check, if the product is properly assembled and all screws are secure and well screwed on.
6. Always check the condition of wear for all the parts to ensure the absolute safe use of the product for people and objects.
7. Periodically check its sturdiness and the stability.

8. Assemble or unfold the stretcher if it is in storage mode.

9. Use of the Product:

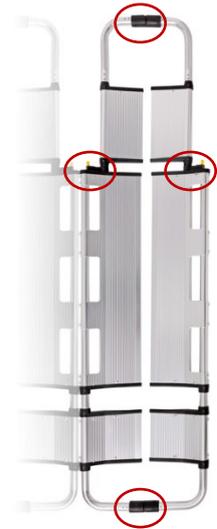1. Only manual lifting and at least two (2) operators are allowed.
2. Lay the stretcher next to the patient and adjust the length to the patient's height.
3. Unlock the couplings at end of the stretcher and separate the stretcher halves.
4. Position half of the stretcher on each side of the patient, aligning the center of the headrest area with the patient's nose.
5. To rejoin the stretcher halves, align the right and left halves of the head and foot couplings and push them together until the locks engage. Use care to avoid pinching or pulling the patient's skin, hair, or clothing while working the stretcher halves into place.
6. Fasten and tighten all patient restraints.
7. Before lifting the stretcher, make sure that the product in its folding parts is secured and you have properly strapped the patient with the safety belts.
8. DO NOT use with patients who exceed the weight limit on the product label and in this user manual.
9. Always inform the patient when adjusting to the stretcher. Never leave stretcher unattended with patient inside

10. Requirements for operators:

- To be trained to deal with corresponding situations.
- Or have practical knowledge of handling patients in emergency situations.
- Be familiar with the procedures and the stretcher.
- Be physically able to assist a patient.

11. Maintenance:

Check the stretcher after each use. The inspection is intended to investigate anything that may compromise the integrity and proper operation of the stretcher. Check the product for any looseness, cracks, breaks, bent or damaged parts.

WARNING

In the event of a problem occurs, DO NOT use the product until the issue will be repaired.

12. Cleaning And Disinfection:

- Clean the parts with water, rinse and wipe well with a soft and clean cloth and dry away from direct sunlight.
- You can use a wet cloth and mild soap on the surfaces that need a better cleaning.
- DO NOT leave the product wet or soiled with dust or mud.
- Do not use chemical cleaners to clean the frame. This may damage the product's coating surface.
- CAUTION: Do not use acidic or caustic solutions, Do not use solvents or stain removers
- If you must disinfect the product, use a common, mild disinfectant, after reading the instructions provided by the disinfectant manufacturer.
- You can sterilize the product at a temperature of up to 100°C.
- The locking mechanisms on the top and rear part of the stretcher may need additional lubrication with use. On a clean cloth put some lubricant and wipe over the mechanisms.
- Protect the product from scratches, cuts and punctures.

13. Conditions of Storage and Transport

The product should be stored packaged under the following conditions:

- For long-term storage the product should be placed in a dry and ventilated place and should not be placed in an environment with high and rapid changes of temperature.
- DO NOT store the product in locations that are close to high temperature sources or direct sunlight for long periods of time.
- The product should be isolated from chemicals and corrosive substances.

- The product should be packaged to protect it from dust, and parts that may be affected or damaged by oxidation should be lightly lubricated.
- During transportation and storage, when packed, the maximum stack height is six layers.
- The product can be transported by any common vehicle.
- Temperature range: -20°C ~ 60°C.

14. **Environmental Protection Statement:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

15. **Incident Reporting:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

16. **Declaration Of Conformity:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

17. **Terms of Warranty:**

The product is guaranteed for a period of two (2) years warranty from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, wheel tire, safety belt, footrests or brakes.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

When the user or the provider apply on warranty request, provide a valid proof of purchase and the completed warranty form, with a photo of the product serial number and a photo or video of the problem that has occurred. In the event the user cannot provide the above documents, if we can confirm that the product or part is still within the warranty period, we may be able to honor the user's request for coverage under warranty, but we are not obligated to do so.

18. Warranty Form

BUYERS DETAILS:	
FULL NAME:	
PHONE:	
PURCHASING DATE:	
INVOICE NUMBER:	
SERIAL NUMBER:	
LOT:	
DEALER DETAILS:	
FULL NAME:	
ADDRESS:	
PHONE:	
DEALER SIGN AND SIGNATURE	

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19. Disclaimer:

The content of the product manual is the intellectual property of MOBIAK S.A. (hereinafter referred to as the "Company") and the copyright is reserved by the Company. Without the written consent of the Company, no one may copy, export or translate the manual or any content included in any form into other languages without the written consent of the Company. The Company carefully compiles the product manual with a sense of responsibility for users but does not guarantee the completeness or correctness of its contents. The product manual, as a purely technical document, does not include implied content or content with oblique reference to third parties. The Company assumes no responsibility for any typographical errors that may cause ambiguities. In the event of direct or indirect loss of information or business interruption due to the product manual or the product information contained therein, the Company shall not be liable.

When using all MOBIAK S.A. products, users must strictly follow the product manual and the instructions for use available on the Company's official website (www.mobiakcare.com) for the handling of the equipment and not violate any warning marked in the product manual when using the equipment. In case of injury to users or other persons, or material damage caused by the handling of the equipment without following the instructions for use or by violating the warnings marked in the product manual, the persons concerned shall bear the respective legal responsibilities.

As the Company continues to improve and update its products, it is essential that you regularly log on to its official website to refer to the most up-to-date instructions and conditions of use or contact your distributor for the latest user manual. The Company reserves the right to revise the product manual without notice.

The Company reserves the right of final interpretation.

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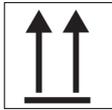
Description of symbols that have been printed on the label, Master Carton and or User Manual

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