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INTENDED USE:

The products are used for quick and effective immobilization of the neck in adult patients & children (depending on the model) with suspected spinal fractures or in routine transports that require preventive immobilization. It can be applied correctly and effectively to people of different body types as it is adjustable, offering the highest possible immobilization of the neck. It can also be used as an aid to relieve pain associated with neck muscle spasms or neck injuries.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- ΜΟΒΙΑΚ S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product.
- ΜΟΒΙΑΚ S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- It is always recommended to follow the procedures designed for emergency response.
- Improper use of the device can cause injury to the patient.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this device or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the device because may be caused damage.
- READ carefully the entire manual before using the product.
- Save this document for future reference.
- The product should be used only by qualified personnel.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play or operate with the device.
- It's important that the patient has been properly immobilized to prevent spine movement.
- Do not make adjustments to the collar while the patient is wearing it.
- Do not store collar in folded position. Please store it in flat position.

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BEFORE EVERY USE:

- Check whether the product is damaged to ensure the safe use of the product (no cracks or breaks in the plastic parts).
- In case of damage, do not use the product and contact the authorized and trained dealer
- Periodically check durability and stability.
- Check for tears in the inner foam material.

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CLEANING AND DISINFECTION:

- Do not machine wash.
- Can be cleaned with mild soap and cold water followed by wiping well with a clean damp cloth to remove all soap residue and dirt.
- If you must disinfect the product, use a common, mild disinfectant
- Protect the product from abrasions, cuts and punctures

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

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INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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USER MANUAL: ADJUSTABLE CERVICAL COLLAR FIRST AID

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MOBIAK S.A