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INTENDED USE:

The product is used for the transportation of patients in stairs, in emergency or in situations demanding a quick and safe evacuation of the patient. It must be used exclusive by professional and qualified personnel, duly trained for emergency care. Can be used in ambulances, in hospital and sports environments.

INDICATIONS:

For the transport of patients, regardless of age & gender, in emergency situations or in situations requiring quick and safe removal of the patient.

Must be used exclusively by specialized and trained personnel. trained in "emergency" cases.

CONTRAINDICATIONS:

It is contraindicated for use by non-specialized and untrained personnel.

Improper use of the device may cause injury to the patient.

If the surface is damaged (cracks or breaks), do not use the product.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration on the frame of the product.
- MOBIAC S.A reserves the right to change the information contained in this document without previous notice
- There is a ± 3%. Deviation for the Dimensions of the Products.
- It is always recommended to follow the procedures designed for emergency response.
- Improper use of the device can cause injury to the patient.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this device or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the device because may be caused damage.
- READ carefully the entire manual before using the product.
- Save this document for future reference.
- The product should be used only by qualified personnel.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play or operate with the device.
- Special care is always required where there are moving parts that can cause limb entrapment or injury.

BEFORE EVERY USE:

- Check if the frame is damaged in order to guarantee a safe use of the product (there are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- Check, if the product is properly assembled and all screws are secure and well screwed on.
- Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
- Periodically check its sturdiness and the stability.

CLEANING AND DISINFECTION:

- Use only a wet cloth and mild soap on surfaces where they need cleaning. Then dry thoroughly before you use the product.
- Dry away from direct sunlight.
- DO NOT leave the product wet.
- Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product.
- Never use acids, alkalis or solvents such as acetone
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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Connect the DC adapter a1 to the battery charging port a3 (connect the other end a2 to the power outlet) for charging and make sure the battery is fully charged before using the stretcher. When the battery indicator a4 turns green from red, it means the battery is fully charged.

- The charger recharges the battery to 100%.
- If the battery level is less than 25%, charging is required.
- Do not disassemble or modify the charger.
- Make sure the charger is not exposed to moisture when charging the battery.
- Never touch the plug with wet hands.
- The battery must be checked and charged before use and at least every 2 weeks.
- The battery must be charged after each use and can be charged on or off the product.

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The slot on the inside of the battery must align with the recess in the stretcher. Push the battery along the arrow b1 until it clicks into place. Turn the key clockwise 180° to lock the battery in the stretcher.

Turn the key counterclockwise 180° and push the battery forward in the direction b2 to remove the battery.

Press the button in the middle, where there are four red lights. Each one means 25% battery level. Do not use the product when only one red light is on. Fully charge the battery and then use it.

When it shows low power or voltage, charging is required.

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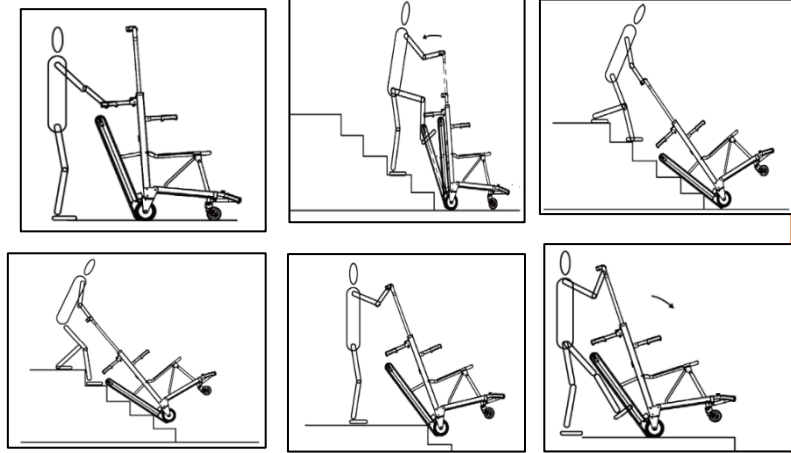
Power on → Turn on the battery switch (I) as shown in f1, press the POWER button - LED display opens - battery remaining display.

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Power off → Press the POWER button, the LED display turns off. Close the battery switch (O) as shown in f2.

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The battery has two keys and they can be used to lock the battery to the unit.

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WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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MOBIAK S.A.