GR
EN

MOBIACARE[®]
MEDICAL EQUIPMENT

MULTILINGUAL
MANUALS

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MEDICAL • GASES • FIREFIGHTING est. 1977

Intended Use:

The Forearm Crutches are walking aids that can be used with one hand and are aimed for people with limited mobility and difficulty for walking, for disable or patients, to help move in and out of the house or and their rehabilitation.

It is recommended to use two pieces at the same time, one in each hand.

They can also be used if an injury or surgery requires you to protect your foot from being loaded with your weight.

Assembling:

The Forearm Crutches are delivered assembled & ready to use.

To adjust the height, press the push button and slide the Forearm Crutches to the desired height. Be sure the push button is fully protruded through the desired hole when finished.

Before use:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check the correct assembly of the device.
3. Check if the safety washer is tightened well.

Maintenance & Cleaning:

- Clean each section of the product frequently.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.
- If you need to disinfect the product, use a common, mild disinfectant.

CAUTION (!):

In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.

Further Instructions:

1. Forearm Crutches handles should be at the height of the patient's hips.
2. Your elbows should be slightly bent when holding the handles.
3. Lean forward slightly and place your Forearm Crutches about 35 cm in front of you. Start your step as if you were going to use your injured leg, but throw your weight on the Forearm Crutches instead of your injured leg. Your body should swing forward.
Complete your step normally with your uninjured foot. When your uninjured foot is on the ground, move the Forearm Crutches forward as part of the preparation for the next step. Focus on the ground, not your feet.
4. For the stairs we follow three simple rules: We go up or down one step at a time. When we go up: The uninjured foot goes up first after the injured and finally the crutches (but on the same step). When we go down: the crutches are placed on the next step followed by the injured leg and finally the uninjured foot (all on the same step)

Environmental Protection:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

Declaration of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

Warranty:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

Warranty Claim Form

Buyers Data

Name:
Date of purchase: **S/N:**
LOT:..... **Phone:**

Resellers Data

Name :
.....
.....
Address:.....
.....
Phone:

Reseller Stamp

INCIDENT REPORTING:

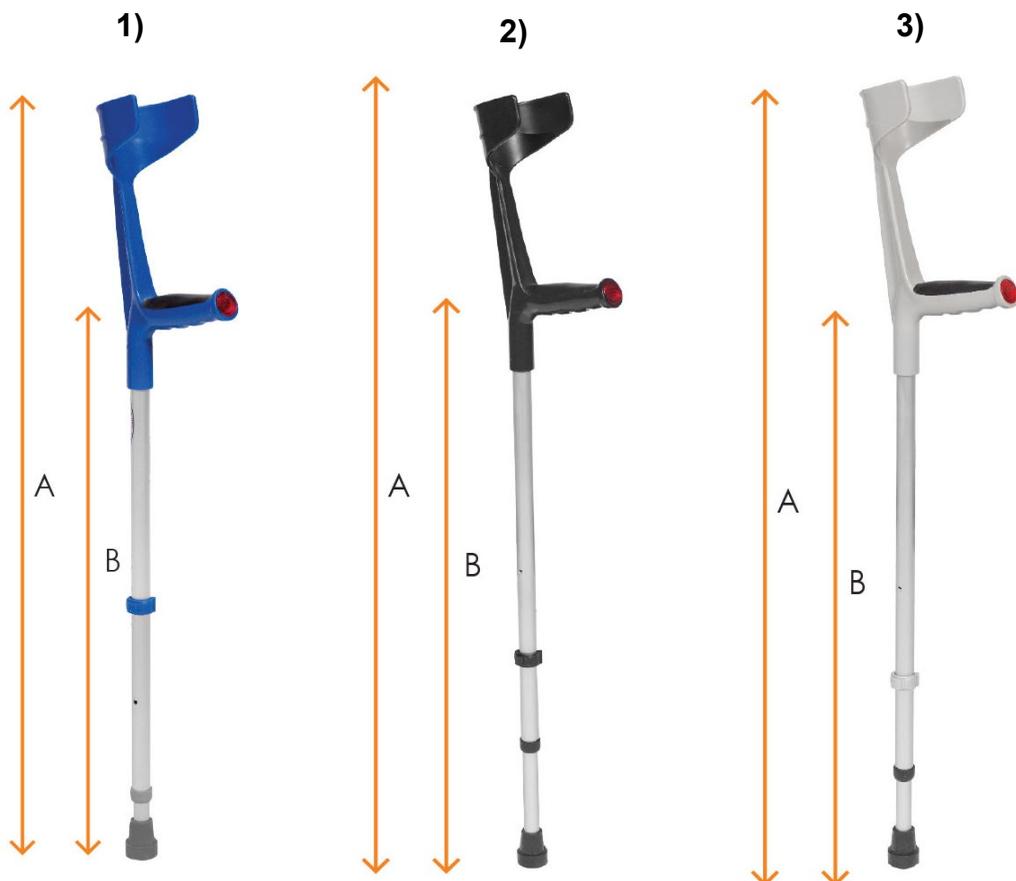
Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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