MOPEDIA®

IMBRACATURE

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INDICE

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Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN ITALY**

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

NOTE

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com

MOPEDIA®

ENGLISH

MA RP893_RP893XL_RP896 02 B_ENG_11 2022

SLINGS

INSTRUCTION MANUAL

INDICE

1. CODES	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNING	PAG.4
6. SYMBOLS	PAG.4
8. BEFORE USE.....	PAG.5
9. WARNING FOR CORRECT USE	PAG.5
10. HOW TO USE	PAG.6
11. MAINTENANCE	PAG.6
12. CLEANING AND DISINFECTION	PAG.6
13. CONDITIONS OF DISPOSAL	PAG.6
14. TECHNICAL FEATURES.....	PAG.7
15. WARRANTY	PAG.8
16. EXEMPT CLAUSES.....	PAG.9

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RP893 Dorsal sling with velcro fastening

RP893 Dorsal sling with velcro fastening XL

RP896 Dorsal sling with velcro fastening & pelvic traction

2. INTRODUCTION

Thank you for purchasing a MOPEDIA sling by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

3. INTENDED USE

The MOPEDIA by MORETTI slings are designed to transport patient with mobility disease.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of STANDAR SLINGS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the sling
- Disinfect the item after every usage
- SERVICE LIFE- the sling use limit is defined by the wear of the parts
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

- Carefully read the stand up lifter manual and follow the instruction suggested on the same
- In case of use of harnesses with a different maximum load from the one provided for the stand up lifter, always use the lowest maximum load.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Maximum washing temperature 30°C; normal process

Do not dry clean

Do not bleach

Do not tumble dry

Do not iron


7. GENERAL DESCRIPTION

Ensure greater toughness at each end is looped with three attachment points. with belt with Velcro for greater support when lifting the patient from a sitting to standing position.

Also available model with pelvic traction for more help in lifting (REF. RP896).


Sling for stand up lifter. Made in 100% polyester fabric (air mesh), strong and breathable, sewn with polyester thread. The border is made by a strong ribbon 100% polypropylene.

The ribbon for the border is continuous and sewn around the entire perimeter of the sling to ensure greater toughness, at each end is looped with three attachment points. Equipped with belt with Velcro for greater support when lifting the patient from a sitting to standing position. Available also a model with pelvic traction for more help during lifting (model REF. RP896).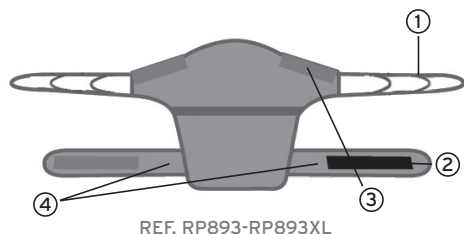

REF. RP893-RP893XL

1. Loops
2. Velcro
3. Padded shoulder rest
4. Belt

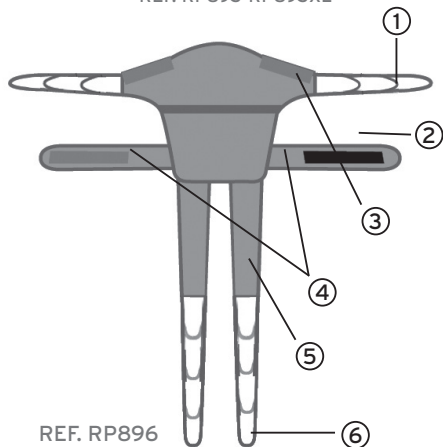REF. RP896

1. Loops
2. Velcro
3. Padded shoulder rest
4. Belt
5. Belt for pelvic traction
6. Loops

8. BEFORE USE

Always verify the sling is in perfect condition, carefully check the seams and the loops.

9. WARNING FOR CORRECT USE

- Before use, make sure the device is not damaged: the tissue, the border, the sewing and suspension loops must be intact
- Periodically check for any damage or usury, every 6 month at least, in particular if you often clean the sling
- Ensure the loops are correctly hooked to the cradle
- Store and use away from flame or source of heat
- Do not exceed the maximum load

- Always consult the user manual of the stand on which you install the harness and strictly follow the instructions given

- Evaluate the risk, in order to guarantee the exact body support unit for the patient, according to the correct dimensions, typology and shape for him
- In case of use of sling with a different maximum load from the one provided for the stand up lifter, always use the lowest maximum load

10. HOW TO USE

The MOPEDIA slings by Moretti can fit all kind stand up lifter.

11. MAINTENANCE

The devices MOPEDIA by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 6 months.

12. CLEANING AND DISINFECTION

Maximum washing temperature 30°C; normal process

Do not bleach

Do not tumble dry

Do not iron

Do not dry clean


13. CONDITIONS OF DISPOSAL


In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the sling in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES


MATERIAL 100% Polyester
DIMENSION Measures in CM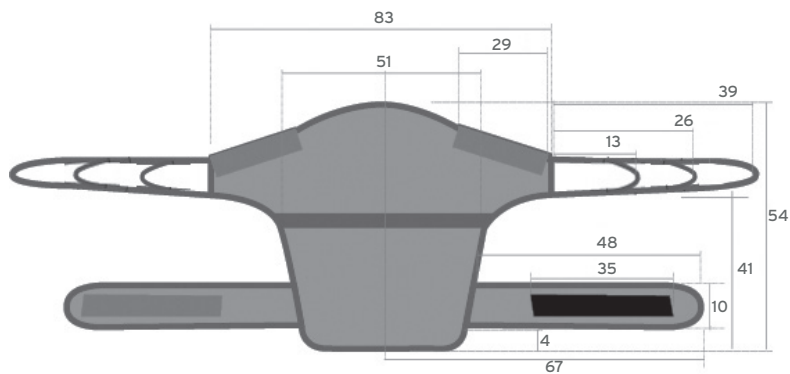

REF. RP893

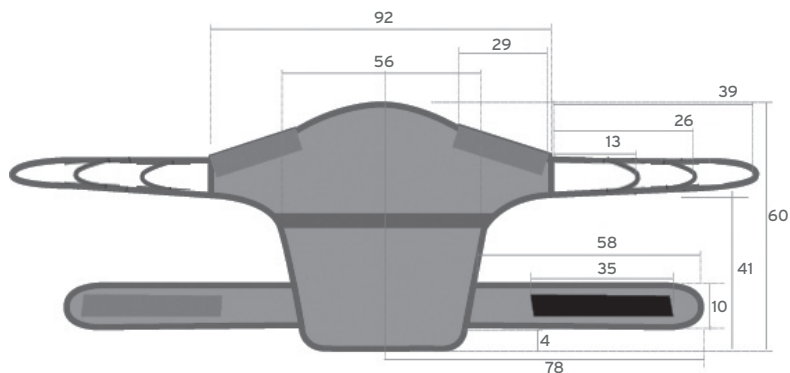REF. RP893XL

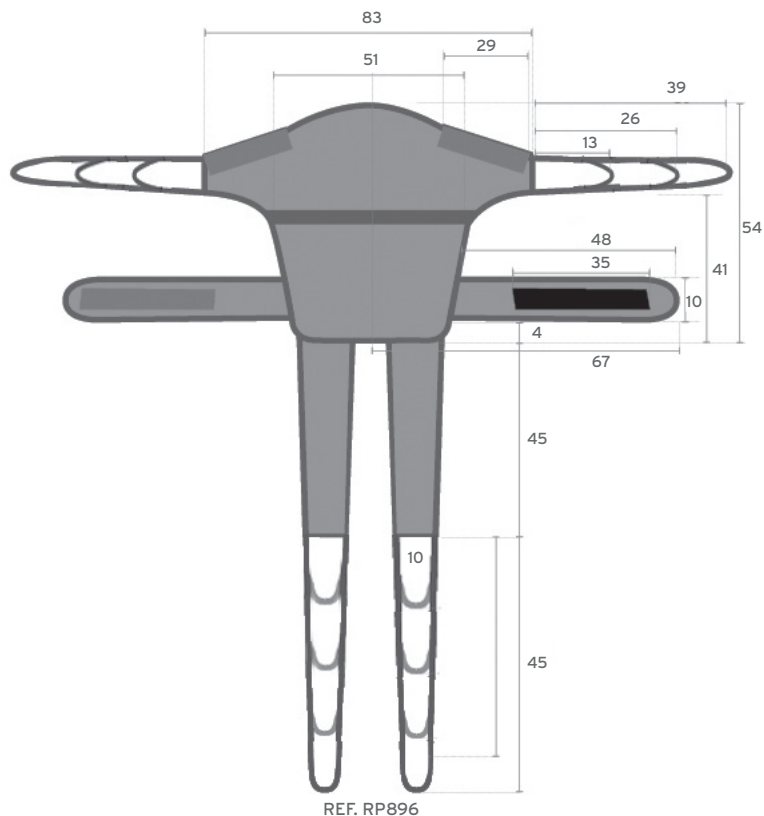15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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